


Initial: 10/10/2007
Reviewed/revised:
Revision:

**MILWAUKEE COUNTY EMS
PRACTICAL SKILL
INFRA RED DATA**

Approved by: Ronald Pirrallo, MD, MHSA
Signature:
Page 1 of 1

UPLOAD FOR ZOLL AED PRO OR AED PLUS

Purpose: To transfer resuscitation information from the Zoll AED Pro or AED Plus to the RescueNet server using infrared ports		Indications: Patients with any Zoll AED Plus or AED Pro monitoring	
Advantages: Captures and analyzes all resuscitation information electronically	Disadvantages: None	Complications: Loss of information if upload procedure not followed correctly	Contraindications: None

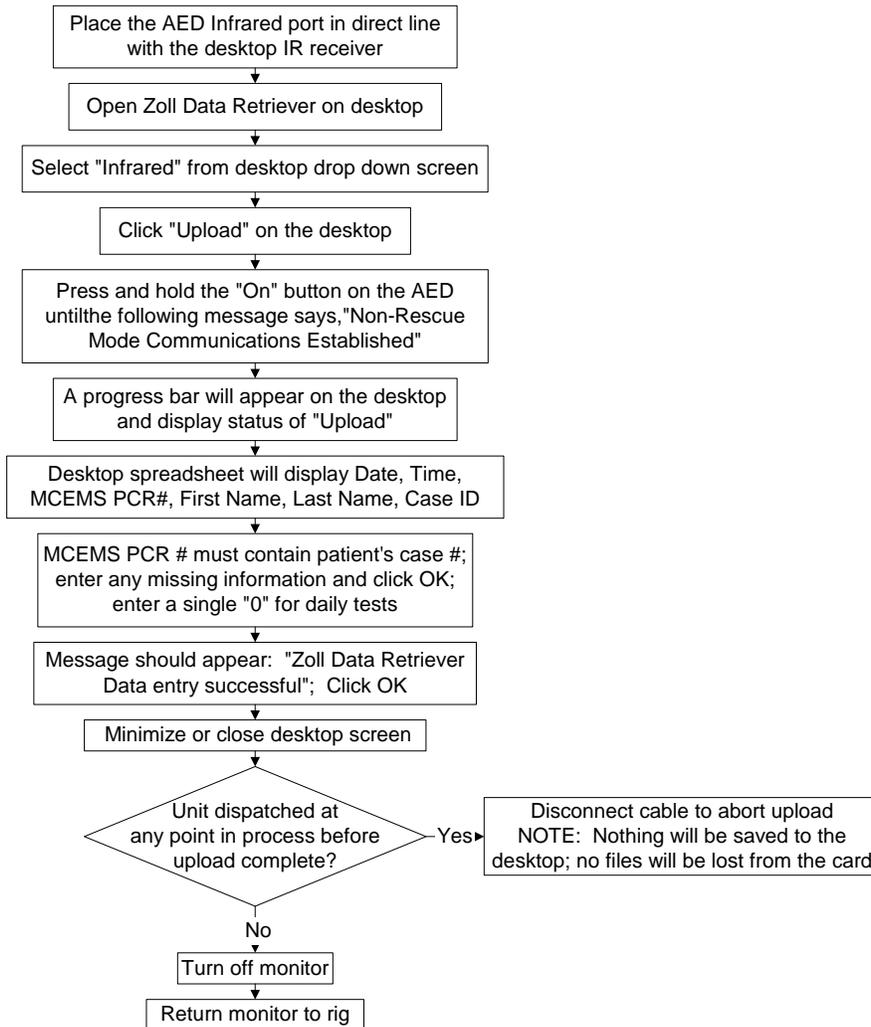

NOTES:

- The MC EMS PCR number must be entered for every case to link the ECG information to the patient's electronic run report. The number can be entered at any time – during the call or at the time of upload
- Enter a single "0" as the MC EMS PCR number for daily tests
- To avoid entering the PCR number numerous times, leave the monitor on and leads attached to the patient during the entire call